

Catchem' POLYP TRAP



ET2211

Catchem' Polyp Trap, 4 Chamber, Removable

ET2213

Catchem' Polyp Trap, 4 Chamber, Fixed

Features and Benefits

Packaged individually

Single-use device minimizes the risk of cross contamination.

In-line installation

Conveniently connects between the endoscope and suction canister.

Clear plastic design

Allows for visual confirmation during polyp collection.

4-Chamber index

Accurately identify each specimen sample in fixed or removable chambers.

Think Within.



Endo-Therapeutics, Inc.
15201 Roosevelt Blvd, Suite 104
Clearwater, FL 33760

EndoTherapeutics.com

Toll Free: **844.400.ENDO** (3636)

Capture excised tissue with a quick and convenient polyp collection and transport container.

ET2211

Catchem' Polyp Trap, 4 Chamber, Removable

Quickly and easily collect tissue samples using the removable chamber design.



ET2213

Catchem' Polyp Trap, 4 Chamber, Fixed

Traditional tissue collection is available using the fixed chamber design.

Instructions

- | | |
|-------------------|---|
| Step One | Connect suction tubing to center port on trap and outside tubing on lid to endoscope. |
| Step Two | Rotate arrow on lid to position tubing over chamber 1. |
| Step Three | Retrieve and collect specimen. |
| Step Four | Rotate lid clockwise to position tubing over chamber 2 after first specimen is collected. |
| Step Five | Retrieve and collect specimen. |
| Step Six | Repeat step 4 and 5 by positioning tubing over chamber 3 & 4. |
| Step Seven | To detach, disconnect suction tubing from suction system and trap tubing from endoscope. |
| Step Eight | After all specimens are transferred to a transport vial, discard remaining portion of trap immediately. |

ET2211 Catchem' Polyp Trap, 4 Chamber, Removable (12/box)

ET2213 Catchem' Polyp Trap, 4 Chamber, Fixed (12/box)



EndoTherapeutics.com • Toll Free: **844.400.ENDO** (3636)
Endo-Therapeutics, Inc. • 15201 Roosevelt Blvd, Suite 104 • Clearwater, FL 33760

EndoT®, Think Within.® and Catchem'® are registered trademarks of Endo-Therapeutics, Inc. ©2015-2018 Endo-Therapeutics, Inc. All rights reserved. MKT-007 Rev B 0118