

ET2211

Catchem' Polyp Trap, 4 Chamber, Removable

ET2213

Catchem' Polyp Trap, 4 Chamber, Fixed

Features and Benefits

Packaged individually

Single-use device minimizes the risk of cross contamination.

In-line installation

Conveniently connects between the endoscope and suction

canister.

Clear plastic design

Allows for visual confirmation during polyp collection.

4-Chamber index

Accurately identify each specimen sample in fixed or

removable chambers.

Think Within.



Endo-Therapeutics, Inc. 15251 Roosevelt Blvd, Suite 204 Clearwater, FL 33760

EndoTherapeutics.com

Toll Free: 844.400.ENDO (3636)

Capture excised tissue with a quick and convenient polyp collection and transport container.

ET2211

Catchem' Polyp Trap, 4 Chamber, Removable

Quickly and easily collect tissue samples using the removable chamber design.





ET2213
Catchem' Polyp Trap, 4 Chamber, Fixed
Traditional tissue collection is available using the fixed chamber design.

Instructions

Step OneConnect suction tubing to center port on trap and outside tubing on lid to endoscope.

Step Two Rotate arrow on lid to position tubing over chamber 1.

Step Three Retrieve and collect specimen.

Step Four Rotate lid clockwise to position tubing over chamber 2 after first specimen is collected.

Step Five Retrieve and collect specimen.

Step Six Repeat step 4 and 5 by positioning tubing over chamber 3 & 4.

Step Seven To detach, disconnect suction tubing from suction system and trap tubing from endoscope.

Step Eight After all specimens are transferred to a transport vial, discard remaining portion of trap immediately.

ET2211 Catchem' Polyp Trap, 4 Chamber, Removable (12/box)

ET2213 Catchem' Polyp Trap, 4 Chamber, Fixed (12/box)



EndoTherapeutics.com • Toll Free: **844.400.ENDO** (3636)

Endo-Therapeutics, Inc. • 15251 Roosevelt Blvd, Suite 204 • Clearwater, FL 33760