



ezSnare

Polypectomy Snare – OVAL


REF ET1800, ET1801, ET1802, ET1803, ET1804, ET1805, ET1806, ET1807



ezSnare

Polypectomy Snare – HEXAGONAL

REF ET1814

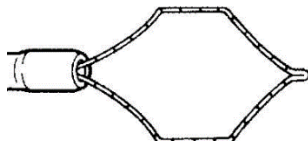

Instructions for Use

Read this document in its entirety prior to use

Rx Only
SINGLE USE, DISPOSABLE

Indications

Polypectomy snares are indicated for use in the removal, via electrocautery, of polyps during endoscopic surgery.

Contraindications

None known within the scope of the indicated use.

Warnings and Precautions

This Device is Sharp! Use same care as for any other surgical device to avoid punctures or cuts to hands, gloves, etc.

Do not use if the sterile packaging is compromised, if any abnormalities are found or if the instrument is not operating properly.

Do not kink tubing when loading into endoscope.

Contact of the snare wire with the endoscope during the procedure may cause grounding, which can lead to injury of the patient and/or operator and damage to the endoscope.

Connection to equipment not complying with EN 60601-2-2 could compromise patient safety.

It is important to use an OLYMPUS® compatible plug. Other plug configurations can lead to faulty operation.

Max peak high frequency output voltage for this accessory device is 4.5 kVp-p.

Instructions

1. Peel open the pouch and remove the device. Visually inspect the device for kinks, bends, breaks or fraying and inspect the handle to ensure there are no broken parts or exposed metal. Check the hot plug and confirm that the prongs are straight and aligned. If any abnormality is found, do not use the device.
2. Inspect the active cord for kinks, bends, breaks and exposed wires. If any abnormality is found, do not use the cord.
3. Confirm the electrosurgical unit is in the OFF position. Attach an Olympus® compatible active cord to the instrument and ensure the connection is tight. The cord should snap onto the handle. If the cord is loose, call your sales representative who can provide you the proper cord.
4. Connect the other end of the cord to the electrosurgical generator and follow the manufacturer's instructions regarding the operation and proper grounding.

NOTE: IT IS IMPORTANT TO USE AN OLYMPUS® COMPATIBLE PLUG. OTHER PLUG CONFIGURATIONS CAN LEAD TO FAULTY OPERATION.

5. Fully retract the snare loop into the sheath prior to inserting the device into the working channel of the endoscope. Use short motions when inserting to avoid kinking.
6. If the endoscope is equipped with an elevator, raise the elevator fully before loading.
7. When the snare is in endoscopic view, extend the loop from the sheath and properly position the snare around the polyp.
8. Following the manufacturer's recommendations, turn on the electrosurgical generator and proceed with the polypectomy.
9. Upon completion of the procedure, turn the electrosurgical generator to the OFF position and then remove the device from the endoscope with the snare loop in the fully retracted position. Failure to fully retract the snare may cause damage to the endoscope.
10. Disconnect the active cord from the device. If the endoscope is equipped with an elevator, it must be lowered before removing the device.

Disposal

Dispose of the device after use in accordance with accepted medical practice and applicable local, state and federal laws and regulations.














Storage

Store sterile instruments at room temperature.


Do not expose to disinfectants, solvents, radiation, UV light or any other substance or environment that could compromise the sterility of the unit.

Do not place any objects on the device or packaging.

Labeling Symbols

	Consult Instruction for Use
	Single use - Do not reuse
	Do not resterilize
	Keep dry
	Ethylene Oxide Sterilization
	Catalog Number / Product Code
	Lot Number
	Use by Date
	Date of manufacture
RxOnly	Federal (USA) Law restricts this device to sale by or on the order of a physician
	Manufacturer
	Quantity of devices in pouch
	Quantity of devices in box
	Do not use if package is opened or damaged



 Endo-Therapeutics, Inc.
 15251 Roosevelt Blvd. #204
 Clearwater, Florida 33760 USA
 +1.844.400.ENDO (3636)
www.endotherapeutics.com