

EndoTBrush

CYTOLOGY BRUSH



FEDERAL (USA) LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN

EN - INSTRUCTIONS FOR USE



Endo-Therapeutics, Inc.
15251 Roosevelt Blvd. Suite 204
Clearwater, FL 33760 USA

EndoTherapeutics.com
Toll Free: **+1.844.400.ENDO** (3636)



Advena Ltd.
Tower Business Centre,
2nd Flr., Tower Street
Swatar, BKR 4013 Malta

Think Within.



REF **ET1420** EndoTBrush, Cytology Brush

REF **ET1421** EndoTBrush, Cytology Brush

REF **ET1422** EndoTBrush, Cytology Brush

REF **ET1423** EndoTBrush, Cytology Brush

Intended Use

Cytology brushes are intended for use in collecting tissue samples for histological examination.

Contraindications

None known within the scope of the intended use.

Warnings and Precautions

Do not use if the sterile packaging is compromised, any abnormalities are found or if the instrument is not operating properly.

Verify brush is fully retracted into outer sheath before pulling device through endoscope channel.

Instructions

1. Peel open the pouch and remove the device. Visually inspect the device for kinks, loose or broken parts. Form the device into an 8-inch diameter loop and deploy and retract brush several times to verify proper actuation.
2. Retract the brush into the outer sheath before inserting the device into the working channel of the endoscope.
3. If the endoscope is equipped with an elevator, raise the elevator fully before loading.
4. If resistance is met when the bending tip of the endoscope is angulated, release the bending angle slightly until the instrument passes smoothly. Do not force the device into an angulated endoscope as this could damage the endoscope.
5. To take a sample, deploy the brush carefully while visualizing the procedure through the endoscope. Move and/or rotate brush tip over biopsy site to collect cells.
6. After cells are collected, retract brush into outer sheath and slowly remove device from endoscope. If the endoscope has an elevator, it must be lowered before withdrawing the device.

Notes

Use of this device restricted to a trained healthcare professional.

In the event of a malfunction, discontinue use and save for return and contact your local representative.

Serious incidents that have occurred in relation to this medical device should be reported to the manufacturer and competent authority in the country where the incident occurred.

Disposal

Dispose of the device after use in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

Storage

Store sterile instruments at room temperature.



Do not expose to disinfectants, solvents, radiation, UV light or any other substance or environment that could compromise the sterility of the unit.

Do not place any objects on the device or packaging.

Symbols

EN - Symbols

A symbol glossary can be found at endotherapeutics.com/symbol-glossary.

	Quantity of devices in pouch
	Quantity of devices in box

Think Within.



Endo-Therapeutics, Inc.
15251 Roosevelt Blvd, Suite 204
Clearwater, FL 33760
USA

EndoTherapeutics.com
Toll Free: **+1.844.400.ENDO** (3636)

EndoT® and Think Within.® are registered trademarks of Endo-Therapeutics, Inc.
©2018-2026 Endo-Therapeutics, Inc. All rights reserved. P/N 151203, Rev. F, March 2026