# SwirlNet

## ENDOSCOPIC RETRIEVAL NET

Rx only

FEDERAL (USA) LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN

EN - INSTRUCTIONS FOR USE  
BG - ИНСТРУКЦИИ ЗА УПОТРЕБА  
CS - NÁVOD K POUŽITÍ  
DA - BRUGSANVISNING  
DE - GEBRAUCHSANWEISUNG  
EL - ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ  
ES - INSTRUCCIONES DE USO  
ET - KASUTUSJUHISED  
FI - KÄYTTÖOHJEET  
FR - NOTICE  
HR - UPUTE ZA UPORABU  
HU - HASZNÁLATI UTASÍTÁS  
IT - ISTRUZIONI PER L'USO  
KK - ПАЙДАЛАНУ НҮСҚАУЛЫҒЫ  
KO - 사용 지침  
LT - NAUDOJIMO INSTRUKCIJOS  
LV - LIETOŠANAS INSTRUKCIJA  
NL - GEBRUIKSAANWIJZING  
NO - BRUKSANVISNING  
PL - INSTRUKCJA OBSŁUGI  
PT - INSTRUÇÕES DE UTILIZAÇÃO  
RO - INSTRUCȚIUNI DE UTILIZARE  
RU - ИНСТРУКЦИИ ПО ПРИМЕНЕНИЮ  
SK - NÁVOD NA POUŽITIE  
SL - NAVODILA ZA UPORABO  
SR - UPUTSTVO ZA UPOTREBU  
SV - BRUKSANVISNING  
UK - ІНСТРУКЦІЯ З ВИКОРИСТАННЯ

Endo-Therapeutics, Inc.  
15251 Roosevelt Blvd. Suite 204  
Clearwater, FL 33760 USA

EndoTherapeutics.com  
Toll Free: **+1.844.400.ENDO** (3636)

Advena Ltd.  
Tower Business Centre,  
2nd Flr., Tower Street  
Swatar, BKR 4013 Malta

Think Within.

EN – English.....	3
BG – Български (Bulgarian).....	3
CS – Čeština (Czech).....	4
DA – Dansk (Danish).....	5
DE – Deutsch (German).....	6
EL – Ελληνικά (Greek).....	6
ES – Español (Spanish).....	7
ET – Eesti (Estonian).....	8
FI – Suomi (Finnish).....	9
FR – Français (French).....	10
HR – Hrvatski (Croatian).....	10
HU – Magyar (Hungarian).....	11
IT – Italiano (Italian).....	12
KK – Қазақша (Kazakh).....	13
KO – 한국어 (Korean).....	13
LT – Lietuvių (Lithuanian).....	14
LV – Latviešu (Latvian).....	15
NL – Nederlands (Dutch).....	16
NO – Norsk (Norwegian).....	17
PL – Polski (Polish).....	17
PT – Português (Portuguese).....	18
RO – Română (Romanian).....	19
RU – русский (Russian).....	20
SK – Slovák (Slovak).....	21
SL – Slovenščina (Slovenian).....	21
SR – Srpski (Serbian).....	22
SV – Svenska (Swedish).....	23
UK – українська (Ukrainian).....	24
Symbols.....	25

## EN – English

<small>REF</small>	<b>ET2010 SwirlNet</b> Endoscopic Retrieval Net, Mini
<small>REF</small>	<b>ET2011 SwirlNet</b> Endoscopic Retrieval Net, Standard Esophageal
<small>REF</small>	<b>ET2012 SwirlNet</b> Endoscopic Retrieval Net, Standard Colonic
<small>REF</small>	<b>ET2013 SwirlNet</b> Endoscopic Retrieval Net, Standard Enteroscope
<small>REF</small>	<b>ET2014 SwirlNet</b> Endoscopic Retrieval Net, Maxi Esophageal
<small>REF</small>	<b>ET2015 SwirlNet</b> Endoscopic Retrieval Net, Maxi Colonic

### Intended Use

SwirlNet is used in the endoscopic retrieval of foreign body, food bolus and excised tissue such as polyps.

### Contraindications

None known within the scope of the intended use.

### Warnings

Do not use if package is opened or damaged.

Visually inspect before use. If any abnormality is detected that would prohibit proper working condition, do not use.

This device is designed for single use only. Attempts to reprocess, resterilize, and/or reuse may lead to device failure and/or transmission of disease.

Patient clinical conditions and type of foreign objects may result in using various retrieval techniques. The technique used is at the discretion of the physician.

Do not kink tubing when loading into endoscope.

Avoid blindly passing the device past any foreign object.

Avoid inadvertently grasping tissues or organs not intended for retrieval.

Do not use to retrieve sharp objects.

Do not deploy handle quickly, in an acute position, when the endoscope is in an extremely articulated or coiled position.

Maintain moderate pressure when passing the trachea to avoid losing the foreign object.

Use caution when removing the foreign object after retrieval. Doing so may result in damage to the net.

### Instructions

1. Verify the endoscope accessory channel size is compatible with the device.
2. Peel open the pouch and remove the device. Visually inspect the device for kinks, loose or broken parts.
3. Form the device into a “U” shape and deploy and retract to verify proper actuation.
4. Retract the device into the outer sheath before inserting the device into the working channel of the endoscope. If resistance is met when the bending tip of the endoscope is angulated, release the bending angle slightly until the instrument passes smoothly. Do not force the device into an angulated scope as this could damage the endoscope.
5. Deploy, position and retract the net to retrieve the foreign object.
6. After the foreign object is collected, retract the device and endoscope while maintaining moderate pressure when passing the trachea to avoid losing the foreign object.

### Notes

Use of this device restricted to a trained healthcare professional.

In the event of a malfunction, discontinue use and save for return and contact your local representative.

Serious incidents that have occurred in relation to this medical device should be reported to the manufacturer and competent authority in the country where the incident occurred.

### Disposal

After use, this device may be a potential biohazard. Dispose of it in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

### Storage

Store in a dry location, away from temperature extremes.

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EndoT®, Think Within.® and SwirlNet® are registered trademarks of Endo-Therapeutics, Inc.
©2019-2026 Endo-Therapeutics, Inc. All rights reserved. 161109, Rev. E, March 2026